

# DermaCol™ 100

## TYPE 1 BOVINE COLLAGEN POWDER WOUND FILLER DRESSING

For the management of exuding wounds

### DESCRIPTION

DermaCol 100 is a Type 1 bovine collagen powder wound filler dressing. DermaCol 100 is topically applied and designed to come into contact with wound exudate. It maintains a moist wound environment at the wound surface.

### INDICATIONS FOR USE

DermaCol 100 is intended for use in the management of partial and full thickness wounds, pressure (stage I-IV) and venous ulcers, ulcers caused by mixed vascular etiologies, venous stasis and diabetic ulcers, 1st and 2nd degree burns, cuts, abrasions and surgical wounds.

### CONTRAINDICATIONS

Do not use on patients with a known allergy or sensitivity to bovine-derived materials.

**DIRECTIONS** (Please use under the guidance of a healthcare professional.)

#### PREPARE WOUND SITE AND SURROUNDING SKIN:

1. Cleanse wound with advised liquid to remove residue according to local infection control protocol. Deep wounds should be well irrigated.
2. Skin around wound should be clean and dry. Use skin barrier prep as needed.

#### TO APPLY:

1. Apply DermaCol 100 in a layer approximately 1/4" thick. Do not pack tightly.
2. Cover with nonadherent dressing.
3. Repeat process as needed.

## CHANGING THE DRESSING:

- Dressing may be left in place for up to 7 days.
- Change the dressing if drainage seeps through the back of the dressing, or if the edge of the dressing becomes loose; or if the condition of the wound or surrounding skin causes concern; or as directed by a qualified healthcare professional.
- Dressing may be moistened to ease removal.
- Carefully remove adhesive dressings by holding the edge of the dressing and slowly pulling it parallel to the skin.
- Dispose of in accordance with local guidance.

## OTHER CONSIDERATIONS:

- DermaCol 100 is intended for short term use up to 4 weeks. For long term use, an assessment by a healthcare practitioner is recommended.


## WARNINGS

- For single use only.
- If excessive redness, pain, swelling, blistering, infection, no sign of healing or other unexpected symptoms occur, discontinue use of the product and seek advice from a qualified healthcare professional.
- Product is sterile and must not be re-sterilized.
- Product remains sterile unless the package is opened or damaged.
- This product has not been studied in pediatric populations and is intended only for use in adult patients.

## OTHER INFORMATION

- Store in a cool, dry storage environment, <25°C.
- Keep away from sunlight.

Caution: Federal (USA) law restricts this device to sale by or on the order of a physician (or properly licensed healthcare practitioner).

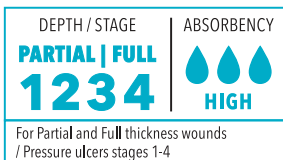

DermaRite®